ardo natal restore

Ardo Natal Restore Supports natural vaginal flora after birth

Read these instructions for use carefully before using the product.

Indications and application: Ardo Natal Restore is a vaginal gel containing pure and natural ingredients for use after childbirth. It moistens the vaginal mucosa and supports the regeneration of the vaginal flora. The gel supports healing and prevents injuries and pain that are caused by dry mucous membranes. It also normalises the vaginal pH. This and the moistening of the mucosa help to prevent infection by bacteria or vaginal yeasts.

Dosage and application: Ardo Natal Restore therapy should start 2 to 3 days after the birth as soon as any birth-related bleeding subsides. The gel is applied to the vagina once a day, preferably in the evening, using the applicator:

- Wash your hands thoroughly before use
- Remove the applicator from the packaging and take off the protective cap
- Insert the tip of the applicator deep into the vagina and apply the entire tube quantity (product is best applied while in a supine position with slightly bent legs)

After application, a slight burning may occur at the beginning of the treatment. This will disappear after a short period of time. The duration of treatment should not exceed 7 consecutive days.

Type of administration: For local application in the vagina.

Precautions: There are no precautionary measures to be taken.

Biocompatibility and use in pregnancy: There are no studies available for pregnant women.

Side effects: There are no known side effects.

Interactions: No interactions have been reported to date.

Contraindications: Ardo Natal Restore should not be used in case of known intolerance to one of the ingredients.

Storage and shelf life: Store Ardo Natal Restore at +5 °C to +30 °C and protected from sunlight. The expiry date is printed on the packaging. Do not use product after the expiry date.

Properties and galenic formulation: Packed in disposable applicator. Contains no medically active ingredients. Colourless gel on a natural basis. The gel is highly viscous, isotonic with an acidic pH value of approx. 4.2.

Dosage form and content: 7 individual applicators of 4 ml each.

Note to the user: According to the Medical Device Regulation, any serious incidents that occur in connection with the product must be reported to the manufacturer and the competent authorities of the Member State in which the user resides.

- Also available:**
- Ardo Natal Perimassage
 - Ardo Natal Support
 - Ardo Natal Anti-Stretch

Explanation of symbols:	
CE ⁰¹²³	CE mark of the notified body
MD	Medical device
⌘	Do not reuse
📖	Observe the instructions for use
LOT	Lot number
EC REP	European representative
⊗	Do not use if packaging is damaged
🔥	Temperature limit
☀	Keep away from sunlight
UDI	Unique device identifier
🕒	Expiry date
🏭	Manufacturer
REF	Item number

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Ardo medical AG Gewerbestrasse 19 6314 Unterögeri Switzerland